## MEDI-KORD A.S.

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## MEDI-KORD A.S.

Braided cotton thread for retraction of movable gingiva with Aluminium Sulfate, for use by dentists only!

MEDI-KORD A.S. is braided cotton cord saturated with a solution of aluminium sulfate and dried by a technique which retains the activity of the active ingredient. Normal tissue moisture reactivates the active ingredient, which promptly retracts gingival tissue in a manner that causes no tissue damage, yet fully exposes the gingival margin.

Available in dispenser of 183 cm of uninterupted thread in 4 sizes: MEDI-KORD A.S. 7, 8, 9, 10.

### COMPOSITION:

The average concentration of aluminium sulfate for each size / cm is:

MEDI-KORD A.S. 7	0.12 mg/cm - 16 H <sub>2</sub> O
MEDI-KORD A.S. 8	0.189 mg/cm - 16 H <sub>2</sub> O
MEDI-KORD A.S. 9	0.335 mg/cm - 16 H <sub>2</sub> O
MEDI-KORD A.S. 10	0.57 mg/cm - 16 H <sub>2</sub> O

The product is not sterile.

### INDICATIONS:

Use whenever gingival retraction is advisable in dental practice, as preceding impression taking with any elastic impression material; or prior to placement of caps, crowns, bridges; and during tooth preparation.

### PRECAUTIONS:

- Read and follow all instructions.
- For professional in-office use only.
- For topical intraoral use only. Do not ingest.
- Keep out of reach of children
- Do not use on patients with aluminium sulfate allergy or sensitivity.

### DIRECTIONS FOR USE:

Use as much cord of the appropriate sizes as is necessary to go all the way around the neck of each tooth to be retracted, with a slight overlap. Two loops may be packed around each tooth (see below), if sulcus space allows.

- For most effective use of MEDI-KORD A.S., loop a strand of smaller cord around the tooth and over the crevice.
- Trim the ends for a slight overlap and pack the loop firmly into the sulcus.

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GB

- Apply a second loop of larger cord over the smaller cord in the same manner.
  - When ready to apply the impression material, remove the larger cord loop only, leaving the smaller cord in place below the gingival margin until the impression procedure has been completed.
  - Once the impression procedure has been completed the smaller cord is removed also.
  - Alternately, a single loop of cord will suffice.
  - MEDI-KORD A.S. completely retracts tissue within five to eight minutes and should then be removed from the mouth.
- Dispensed portions of cord are intended for single use only and should be disposed of immediately after use. Reusing dispensed portion puts patients at increased risk for injury or infection.

### CONTRAINDICATIONS:

There are no known contraindications for the use of MEDI-KORD A.S.. The chemical components of MEDI-KORD A.S. have a long history of safety and effectiveness for the purposes intended.

For product SDS information please see company website at [www.lamaisondentairea.com](http://www.lamaisondentairea.com) or contact La Maison Dentaire directly.

CH REP CH-REP/Importer  
Medirel SA  
via Redondello 20, CH-6982 Agno  
Tel. +41 (0)91 604 54 34 - info@medirel.ch

expiry date is printed on the label